1. COMPARISON VENTO VS FLUTTER (experimental data)

Resistance 1 (R=1, small 'low resistance' ball); Resistance 1 (R=2, standard ball)

Table 1: Mean PEP (cm H₂0)

	0 ⁰ Angle			30 ⁰ Angle		20 ⁰ Angle
	Resolve		Flutter ^a	Resolve		Flutter
Flow (L/min)	Resistance 1	Resistance 2		Resistance 1	Resistance 2	
5	4.9 ± 0.0	6.0 ± 0.0	6.2 ± 0.02	9.6 ± 0.2	11.1 ± 0.1	-
15	5.4 ± 0.1	6.5 ± 0.1	7.1 ± 0.03	9.3 ± 0.0	10.9 ± 0.1	-
25	6.3 ± 0.1	7.7 ± 0.1		10.7 ± 0.1	12.4 ± 0.1	11.5 ± 0.05 ^b

Table 2: Peak PEP (cm H₂0)

	0 ⁰ Angle			30 ⁰ Angle		20 ⁰ Angle
	Resolve		Flutter ^a	Resolve		Flutter
Flow (L/min)	Resistance 1	Resistance 2		Resistance 1	Resistance 2	
5	9.6 ± 0.1	10.9 ± 0.1	9.3 ± 0.16	12.8 ± 0.1	15.2 ± 0.2	-
15	18.0 ± 0.0	20.0 ± 0.0	19.3 ± 0.01	18.8 ± 0.0	22.2 ± 0.1	-
25	26.8 ± 0.1	29.3 ± 0.1		26.8 ± 0.0	31.4 ± 0.2	21.8 ± 0.12^{b}

Table 3: Amplitude PEP (Peak – Nadir)


	0 ⁰ Angle			30 ⁰ Angle		20 ⁰ Angle
	Resolve		Flutter ^a	Resolve		Flutter
Flow (L/min)	Resistance 1	Resistance 2		Resistance 1	Resistance 2	
5	8.5 ± 0.1	9.1 ± 0.1	5.9 ± 0.7	5.9 ± 0.4	7.2 ± 0.1	-
15	17.6 ± 0.0	19.4 ± 0.1	17.9 ± 0.8	16.5 ± 0.1	19.8 ± 0.3	-
25	24.1 ± 0.8	27.5 ± 0.5		24.9 ± 0.1	29.6 ± 0.2	25.0 ± 2.1 ^b

Table 4: Oscillation Frequency (Hz)

0 ⁰ Angle				30 ⁰ Angle		20 ⁰ Angle
	Resolve		Flutter ^a	Resolve		Flutter
Flow (L/min)	Resistance 1	Resistance 2		Resistance 1	Resistance 2	
5	28.8 ± 0.1	30.1 ± 0.1	25.9 ± 0.9	37.3 ± 0.5	35.6 ± 0.1	-
15	19.6 ± 0.1	20.8 ± 0.1	20.0 ± 0.0	30.3 ± 0.0	30.9 ± 0.2	-
25	16.8 ± 0.6	17.6 ± 0.3		24.8 ± 0.3	25.7 ± 0.1	31.3 ± 0.2 ^b

^aFlutter Data collected in January 2021

^b Data extracted from Franks et al, Respiratory Care 2019. 64(4), 434-444.

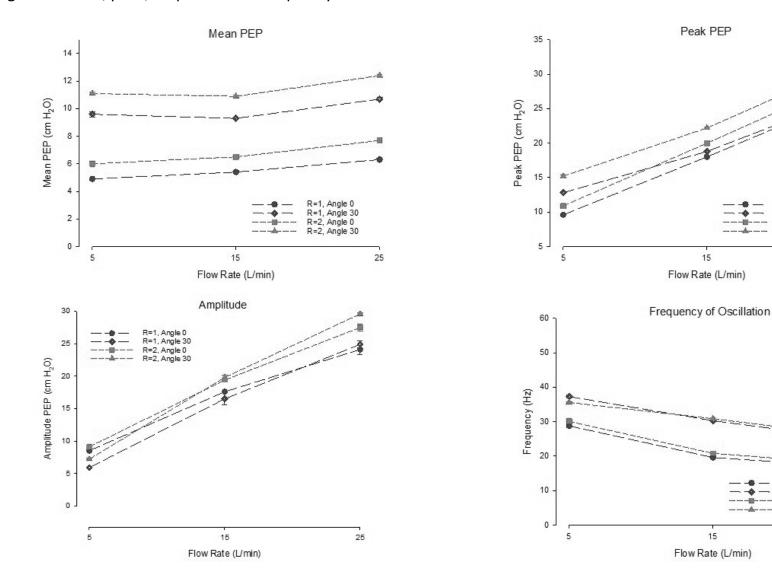

R=1, Angle 0 R=1, Angle 30 R=2, Angle 0

R=2, Angle 30


R=1, Angle 0 R=1, Angle 30 R=2, Angle 0 R=2, Angle 30

25


25

Figure 1. Mean, peak, amplitude and frequency of PEP across flow rates and with two resistances.


Data Collection Images

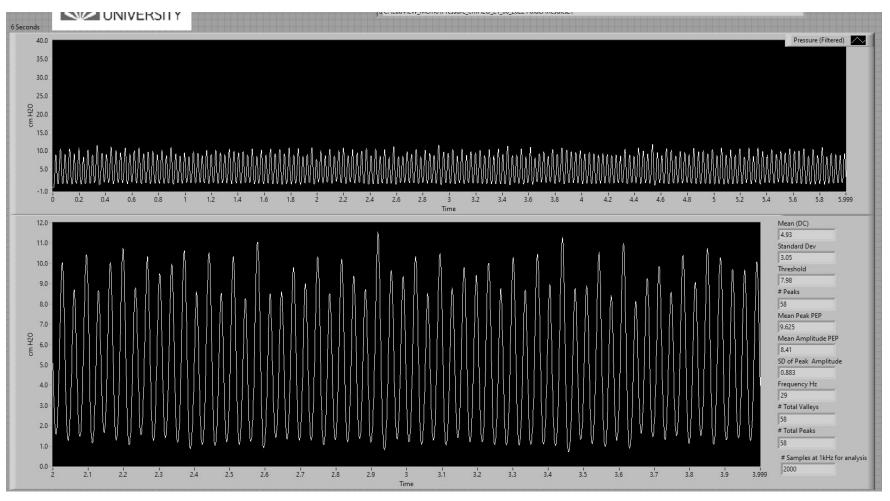


Resistance =1 (small 'low resistance' ball); Flow: 5 L/min; 0⁰ Inclination

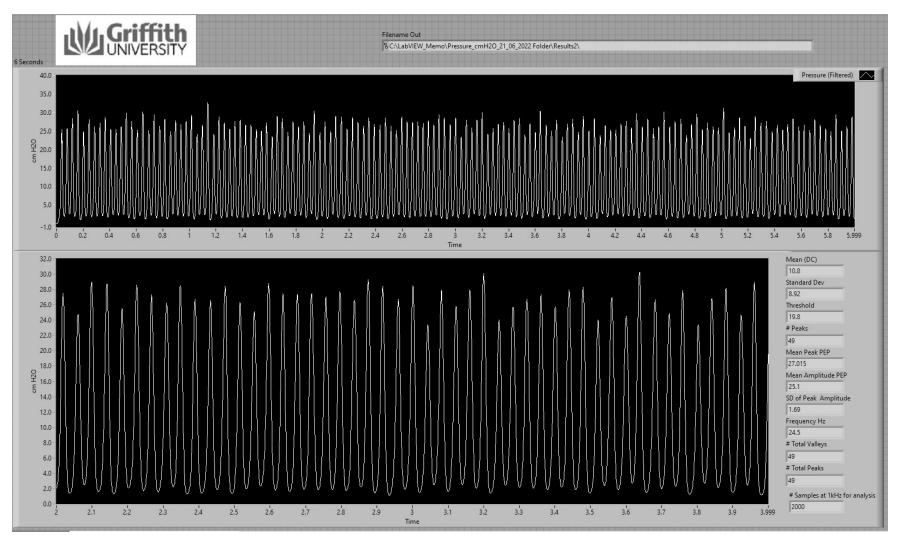

Resistance =1 (small 'low resistance' ball); Flow: 5 L/min; 30⁰ Inclination

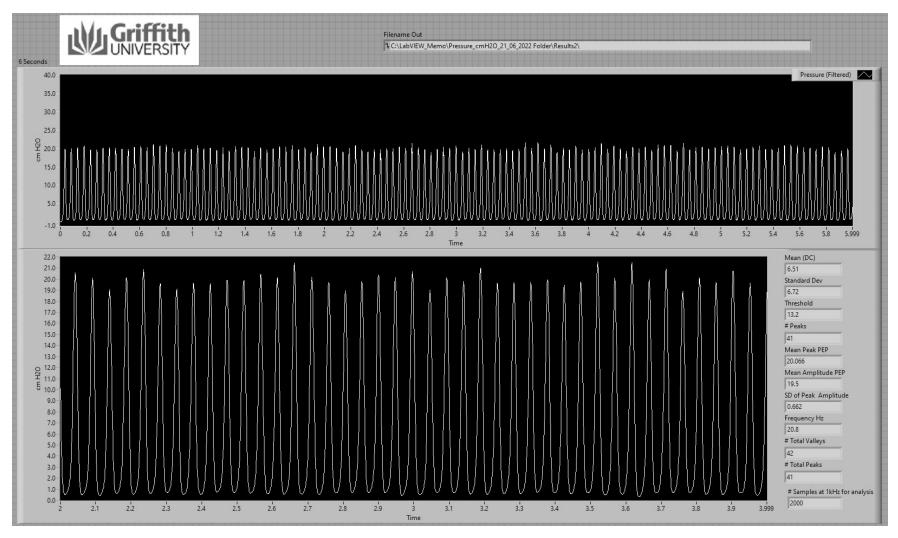

Resistance =2 (standard ball); Flow: 15 L/min; 0⁰ Inclination

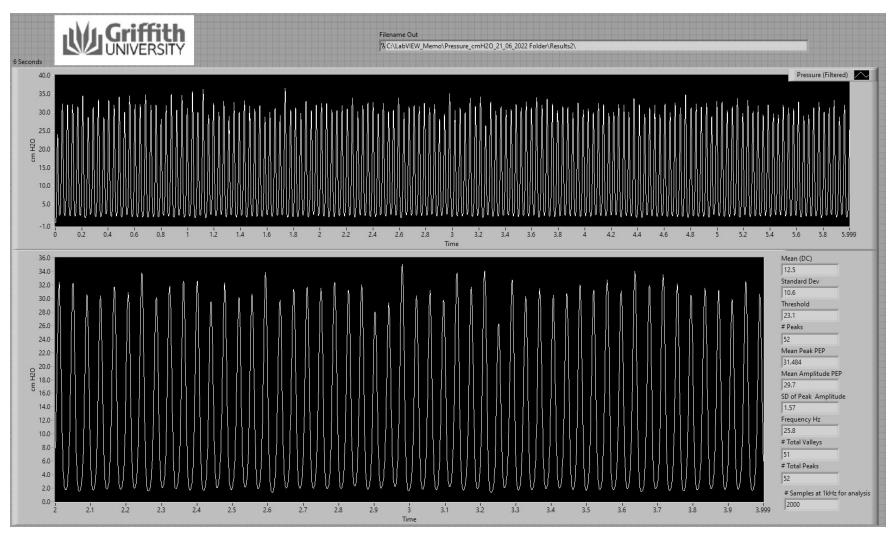

Resistance =2 (standard ball); Flow: 15 L/min; 30⁰ Inclination

Data Publication Note

This report (including any enclosures and attachments) has been prepared for a specific purpose for a particular client. Use of this report for any other purpose (including raising capital) by the particular client or for any purpose whatsoever by any third party is not permitted. This report is not intended for use by a third party and a third party may not place any reliance on this report whatsoever. Griffith University makes no warranty, express or implied as to the accuracy or completeness of the report or any information used in compiling the report. To the extent permitted by law, Griffith University disclaims all liability for any loss or damage of any nature whatsoever which may be suffered by any fault or negligence of Griffith University or otherwise. Griffith University owns or has a right to use the copyright and all intellectual property in the report. No part of this report may be reproduced for any commercial use or distributed or communicated to any third party, without the prior written consent of Griffith University.